

## STEP-BY-STEP IMPORT INSTRUCTIONS

### Indirect import via 3Shape Dental System Control Panel

- **Download the library:**

Download the current Z-SYSTEMS library (Z-Systems Abutment Library\_) from the official Z-SYSTEMS website <https://www.zsystems.com/download>

- **Open the 3Shape Dental System Control Panel**

- **Navigate to :**

Tools  Import/Export  Import material

- **Select the downloaded file 'Z-Systems Abutment Library\_'. Confirm the import and then close the 3Shape Dental System Control Panel.**

**The library has been successfully installed and contains the following system components:**

Article	Article name
Z5-Bone Level	(Z5-BL Abutment and Analog - Connection)
Z5-Tissue Level	(Z5-TL Abutment and Analog - Connection)